


Make sure to Remove the Retainers

Precautions

- Three retainers are attached to this machine to protect it from vibration during shipment. After installing this machine, remove all the retainers. Switching the power on with retainers attached to this machine may cause faulty operation or a breakdown.
- The retainers will be required when moving the machine to a different location. Store them carefully so that they do not get misplaced.

How to Remove

The following image shows the positions where retainers are attached. For details on how to remove retainers, see "Removing the Retainers" in the Setup Guide.

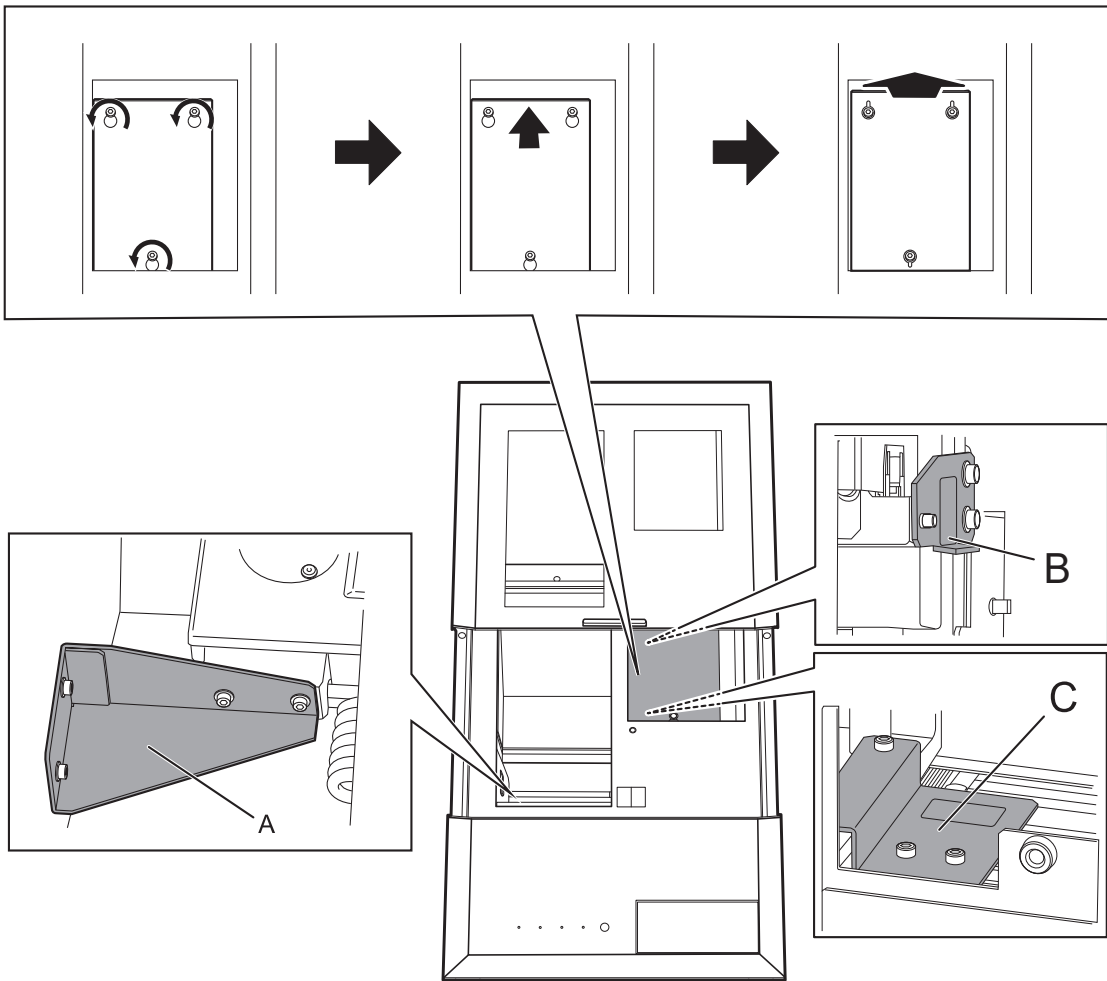

Company names and product names are trademarks or registered trademarks of their respective holders.

<https://www.dgshape.com/>

Copyright © 2023 DGSHAPE Corporation

R1-231228T



1000028859

固定具取り外しのお願い

注意事項

- 輸送の振動から本体を保護するため、本機には3つの固定具が取り付けられています。機体の設置が済んだら、すべての固定具を取り外してください。固定具が機体に取り付けられたまま電源をオンにすると、誤動作や故障の原因になります。
- 固定具は移送の際に必要となります。なくさないよう保管してください。

固定具の取り外し

下図は、固定具が取り付けられている位置を示しています。固定具の取り外し方については、セットアップガイドの「固定具を取り外す」を参照してください。

